RPN 75608

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Installation, operation and maintenance Instructions Stretcher

****WARNING****

These instructions must be provided to the user(s) of this equipment. These instructions must be read, understood, and followed. Failure to do so could result in serious injury or death. Users should be trained in the use of this equipment prior to use. Check with local authorities for regulations governing the use of fall protection equipment.

RPN 75608

“Combining modern technology with old fashioned values” is our corporate statement. It reflects a commitment to excellence by every member of the J.E. Lortie Co. Ltd. team. It extends from the manufacturing personnel to the warehouse personnel, from the beginning to the end. We manufacture products designed to meet or exceed current CSA specifications, as well as proposed ones. We are constantly aware of new methods, and new standards. Most importantly, our products meet our specifications, which are often more rigorous than CSA.

J.E. Lortie Co. Ltd. is committed to quality in every aspect of our business. This commitment begins with the design of the product and the selection of components. It continues through manufacturing, inspection, shipping and customer service. Quality is the number one word at J.E. Lortie Co. Ltd. It is a commitment to excellence that never ends. It is our way of life.

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Components:

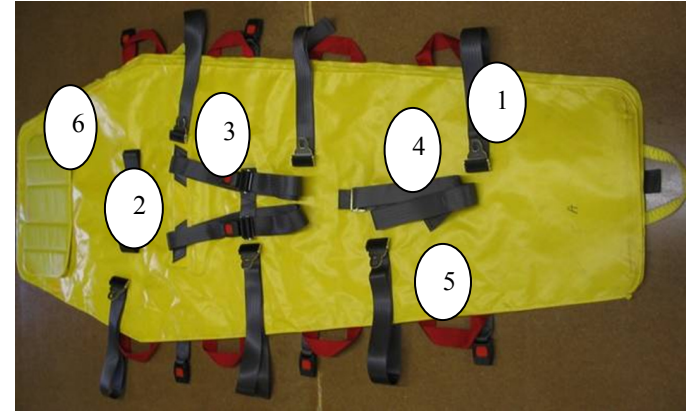
Figure 1: Stretcher

- | | |
|---------------------|-------------------|
| 1- Restraint straps | 4- Crotch strap |
| 2- Chest strap | 5-Carrying straps |
| 3- Arms straps | 6- Pillow |

Figure 2: Head restraint straps

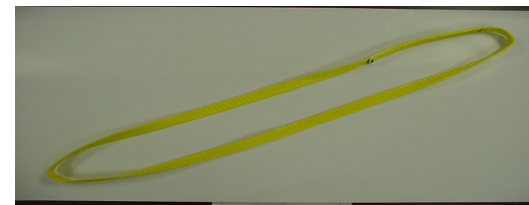
Figure 3: Lifting/Carrying sling

Figure 1: Civière

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Figure 4: Carrying bag

Use and Purpose:

The stretcher should be used in the rescue of a casualty or a transfer, under conditions where it's always loaded.

Models Covered by These Instructions:

UPC 05990

Inspections:

Before each use, inspect the following points:

Textiles

Cuts, laceration, abrasion, damages resulting of deterioration, exposure of the heat, acids and corrosives.

Sewing

Break, cut or used worn

Buckles

The plastic and metal buckles must be carefully examined, at the opening and closing. Also, check if there are any burrs or oxidation on the metal.

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Stretcher

Check the mechanics of restraining straps if they are working. Make sure that the stretcher is clean.

Any stretcher that was damaged must be removed from service

Maintenance and Storage:

J.E. Lortie's stretchers must be kept free of grease, dirt, and oil. All equipment should be stored in a cool dry place and should not be exposed to harsh weather conditions. Do not store this product in direct sunlight. J.E. Lortie's stretchers should only be cleaned with a mild detergent and warm water, toweled off to remove excess water, and then hung to dry. Never place these products in a dryer or expose to excessive heat. J.E. Lortie's stretchers are not repairable. If any part of the stretcher is damaged, or if the stretcher itself is damaged, it must be removed from field service.

Procedures:

Open up stretcher and lay out completely on a flat surface.

Remove all supporting accessories/components (Head restraint straps, lifting slings, etc.)

Undo all straps and prepare for use.

Lay the victim out on the stretcher with great care.

To localize the victim's neck and head, support by raising the head flaps on each side.

Use the Velcro attaching accessory head straps on the forehead and chin to stabilize the neck and head.

Continue by attaching all restraint straps; torso, arms and lower extremities.

Adjust all attaching straps based on the size of the victim.

Prepare to evacuate the victim by lifting using the side web handles. There are 4 handles on each side to facilitate the lift, with the use of one or two people lifting per side.

Use the 4 sling straps where and when necessary.

The stretcher can also be used to lift a victim in the upright/standing position, if necessary by using the web strap handle, located at the extreme end of the stretcher (neck and head end)

Transport the victim to where he/she can get the adequate follow up care.

Training:

Employers are required to provide proper training to all employees by a 'Competent Person' on recognizing fall hazards and how to minimize those hazards, fall protection equipment and its use, the employees role in fall protection, and OSHA standards relating to fall protection (OSHA 1926 Subpart M). It is not necessary to reserve this product to one person; however each person must be trained in the proper use of this product.

Product Markings

The stretcher provides information on the manufacturer, date manufactured, serial number, model number. This label must never be removed from the product. If the label is missing or is illegible, do not use the stretcher.

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Warnings:

- Do not alter this equipment from the original manufacturer's design.
- Do not use this equipment in any way other than its intended use as listed in these instructions. Some chemicals may cause deterioration to this equipment. Contact the manufacturer if there is any doubt about the effect of a particular chemical on this product.

OSHA 1926.32 defines a Competent Person and Qualified Person as follows:

“(f) “Competent person” means one who is capable of identifying existing and predictable hazards in the surroundings or working conditions which are unsanitary, hazardous, or dangerous to employees, and who has authorization to take prompt corrective measures to eliminate them.”

“(m) “Qualified” means one who, by possession of a recognized degree, certificate, or professional standing, or who by extensive knowledge, training, and expertise, has successfully demonstrated his ability to solve or resolve problems relating to the subject matter, the work, or the project.”

This product should be inspected by a competent person at a minimum of every six months.

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WARNING: you must be able to read and understand these instructions to use this equipment. These instructions are in English and in French. If you need a set of instructions in another language, please contact your supervisor, or your distributor, or J.E. Lortie Co. Ltd. for the appropriate instructions.

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J.E. Lortie, Inc.